



IPS EMPRESS[®] 2


Your Bridge ² The Esthetic Revolution.SM



Anterior Chairside Preparation Guide


Full-Coverage Restorations

ANTERIOR CROWN PREPARATION

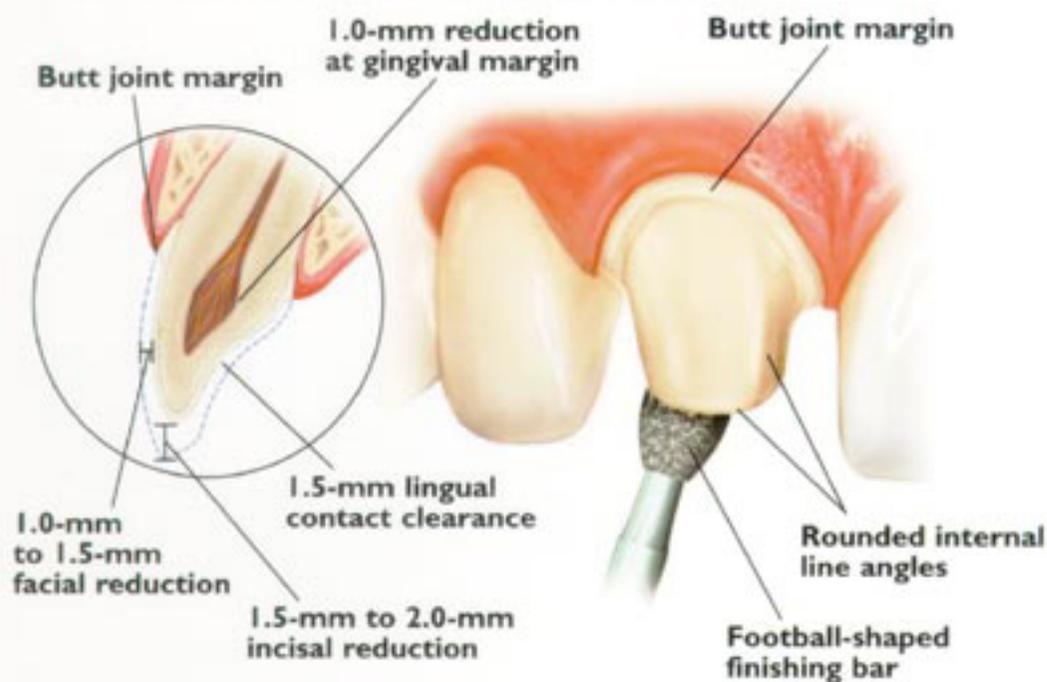

3-Unit Bridge Restorations


3-UNIT BRIDGE PREPARATION

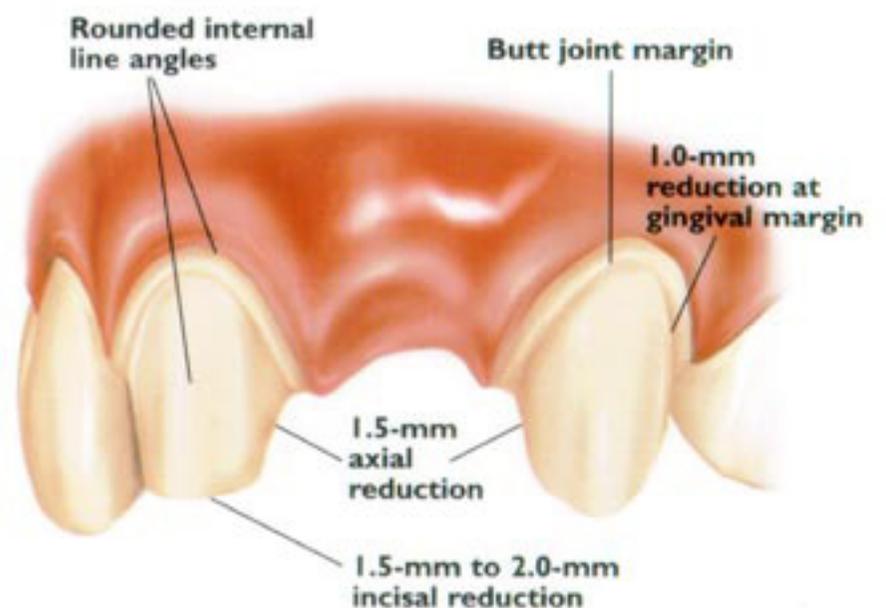

Veneers


UNIFORM FACIAL PREPARATION

A medium grit, round-ended, diamond bur is used to remove a uniform thickness of facial enamel by joining the depth-cut grooves.

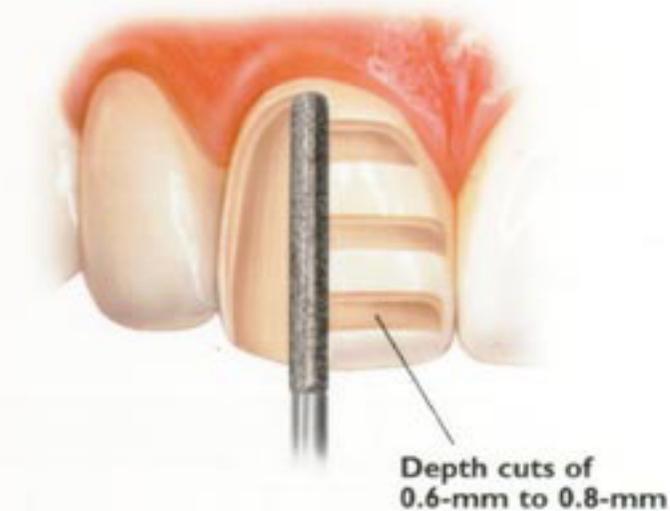

INCISAL PREPARATION

The diamond bur is angled to bevel back the incisal edge.

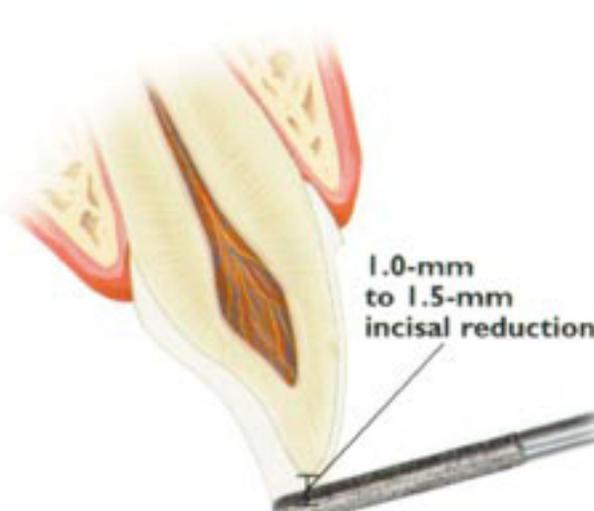



CHAMFER MARGINS

Correct preparation of the chamfer margins interproximally allows the appropriate bulk of porcelain.




3-Unit Bridge Restorations

3-UNIT BRIDGE PREPARATION

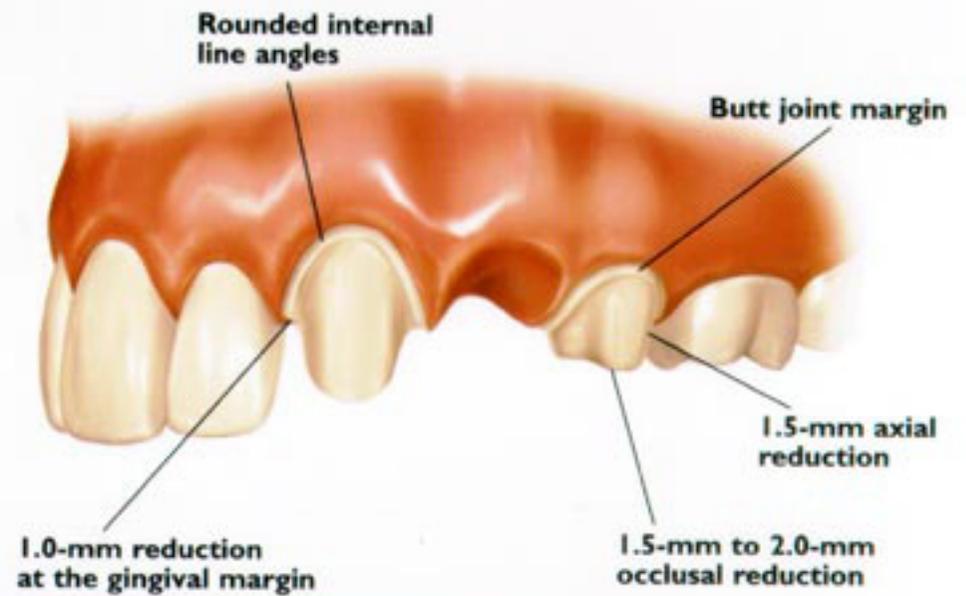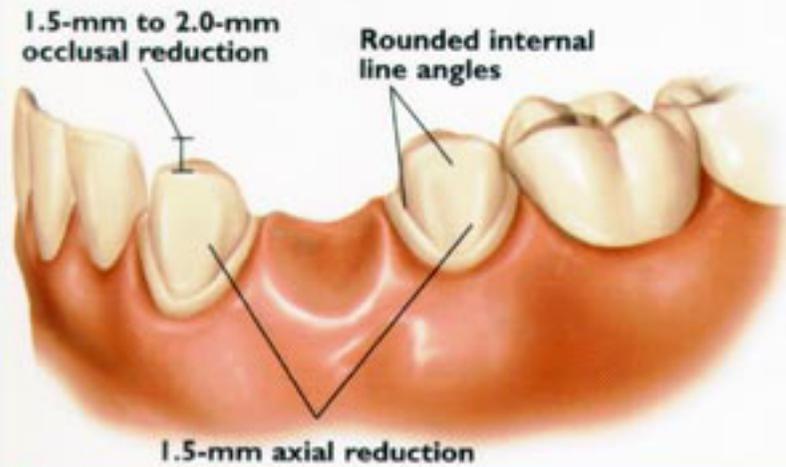


Full-Coverage Restorations

POSTERIOR CROWN PREPARATION

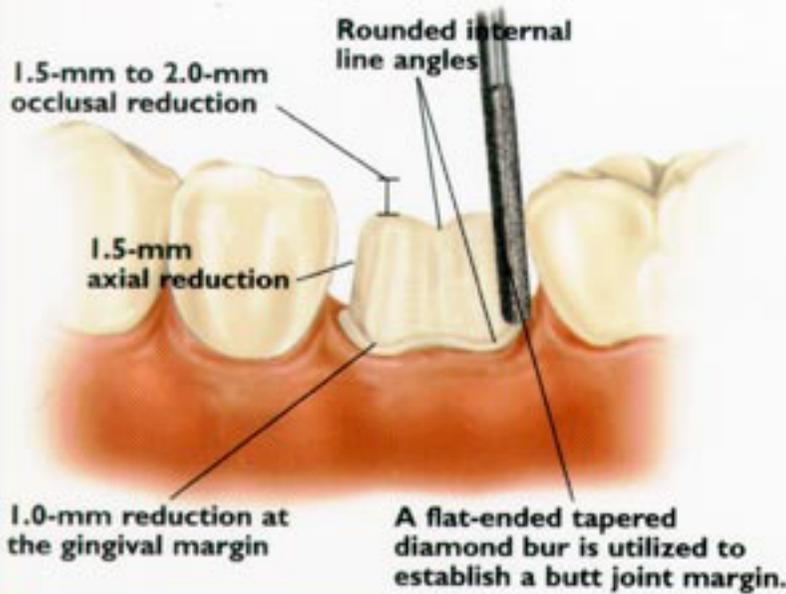


CONVENTIONAL CEMENTATION PREPARATION

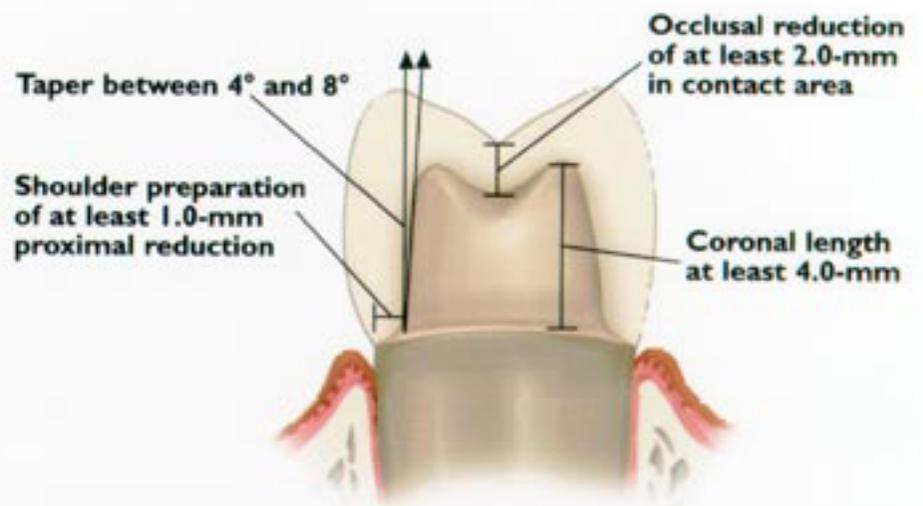

Inlays/Onlays

INLAY PREPARATION

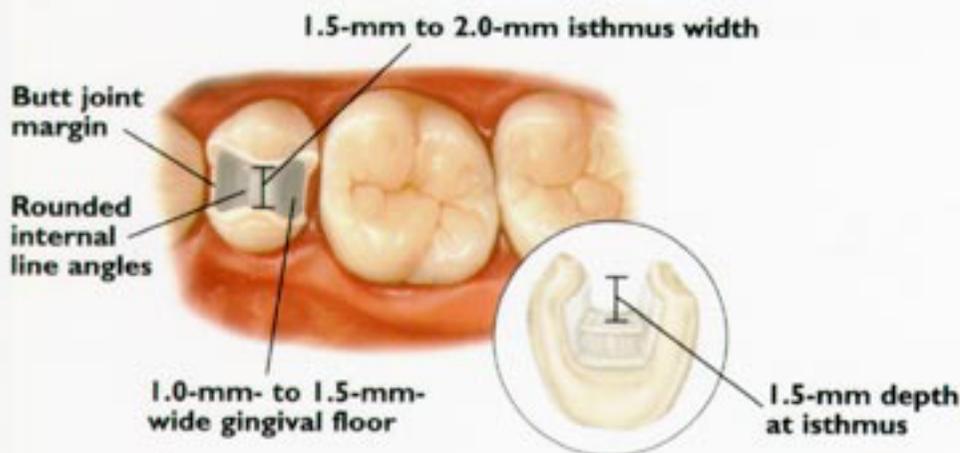

ONLAY PREPARATION

